


D-Pen™

Oral Fluid Drug Test

- Rapid saliva collection
- Consistent rapid wicking
- Up to 12 customizable drugs plus ALCOHOL on one device
- Greatly decreases ability to adulterate sample
- Excellent in a "reasonable suspicion" scenario
- Available for Forensic Use, Employment & Insurance

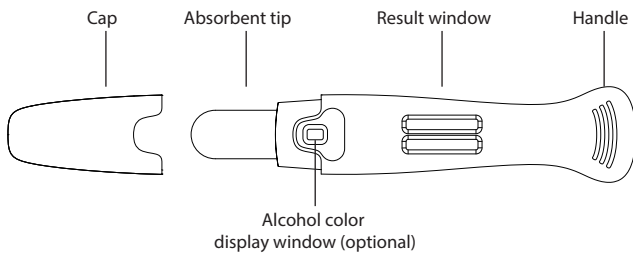

Directions for use

- * When sampling, gently hold it in mouth and let oral fluid naturally adsorb on the absorbent tip.
- * Do not eat, drink, or smoke for at least 30 minutes prior to sample collection.
- * Any oral fluid specimen is appropriate for testing but the oral fluid specimen collected in the morning, before mouth rinsed, eating or drinking, is recommended.

Catalog Number	Combo Items	Qualification
DPO-156EUO	D-Pen™ 5 Panel Oral Fluid Drug Test – AMP 50, COC 20, MET 50, OPI 40, PCP 10	EUO
DPO-256EUO	D-Pen™ 5 Panel Oral Fluid Drug Test – AMP 50, COC 20, OPI 40, MET 50, THC 40	EUO
DPO-496NTEUO	D-Pen™ 9 Panel Oral Fluid Drug Test - AMP 50, BAR 20, COC 20, MDMA 50, MET 50, MTD 30, OP1 40, OXY 20, PCP 10	EUO
DPO-10106EUO	D-Pen™ 10 Panel Oral Fluid Drug Test – AMP 50, BAR 20, COC 20, MDMA 50, MET 50, MTD 30, OPI 40, OXY 20, PCP 10, THC 40	EUO
DPO-266	D-Pen™ 6 Panel Oral Fluid Drug Test –AMP 50, BZO 30, COC 20, MET 50, OPI 40, THC 40	FUO
DPO-276	D-Pen™ 7 Panel Oral Fluid Drug Test –AMP 50, BZO 30, COC 20, MET 50, OPI 40, OXY 20, THC 40	FUO
DPO-186	D-Pen™ 8 Panel Oral Fluid Drug Test –AMP 50, BAR 20, BZO 30, COC 20, MTD 30, OPI 40, PCP 10, PPX 25	FUO
DPO-196PA	D-Pen™ 9 Panel Oral Fluid Drug Test –AMP 50, BAR 20, BZO 30, COC 20, MTD 30, OPI 40, PCP 10, PPX 25, THC 40, +ALC (0.02%)	FUO
DPO-1116PA	D-Pen™ 11 Panel Oral Fluid Drug Test –AMP 50, BAR 20, BUP 5, BZO, COC 20, MET 50, MTD 30, OPI 40, OXY 20, PCP 10, THC 40, +ALC (0.02%)	FUO
DPM-COT-10B	D-Pen™ Mini Oral Fluid Drug Test–COT (50ng/ml)	FUO
DPM-THC-10B	D-Pen™ Mini Oral Fluid Drug Test–THC (40ng/ml)	FUO

Available Test Items

Specimen	Test Item	Cut-off level	Format	Qualification
Oral Fluid	Amphetamine (AMP)	50/40 ng/mL	Variable	EUO/FUO
	Barbiturates (BAR)	60/20 ng/mL	Variable	EUO/FUO
	Cocaine (COC)	50/20 ng/mL	Variable	EUO/FUO
	Marijuana (THC)	50/40/12 ng/mL	Variable	EUO/FUO
	Methadone (MTD)	30 ng/mL	Variable	EUO/FUO
	Methamphetamine (MET/mAMP)	50 ng/mL	Variable	EUO/FUO
	Methylenedioxymethamphetamine (MDMA)	100/50/25 ng/mL	Variable	EUO/FUO
	Opiate (OPI)	50/40/15 ng/mL	Variable	EUO/FUO
	Oxycodone (OXY)	20 ng/mL	Variable	EUO/FUO
	Phencyclidine (PCP)	10 ng/mL	Variable	EUO/FUO
	Benzodiazepines (BZO)	30/10 ng/mL	Variable	FUO
	Buprenorphine (BUP)	5 ng/mL	Variable	FUO
	Cotinine (COT)	20/50 ng/mL	Variable	FUO
	Ethyl Glucuronide (ETG)	300/500 ng/mL	Variable	FUO
	Fentanyl (FTY)	20 ng/mL	Variable	FUO
	Morphine (MOP)	15 ng/mL	Variable	FUO
	Monoacetylmorphine (6-MAM)	10 ng/mL	Variable	FUO
	Propoxyphene (PPX)	25 ng/mL	Variable	FUO
	Alcohol	0.02%	Variable	FUO

**Customization available upon request.*