

Amazewell™

Digital Pregnancy Rapid Test

INSTRUCTIONS FOR USE

PLEASE READ ALL INFORMATION IN THE INSTRUCTIONS FOR USE BEFORE USING THE TEST!

Model: DC081A

Catalogue No. See Box label

Amazewell™ Digital Pregnancy Rapid Test is intended for the qualitative detection of human chorionic gonadotropin (hCG) in urine, as an aid in early detection of pregnancy. It is intended for use by people who would like to find out whether they are pregnant in a home environment.

For external use only. For over the counter use.

CONTENTS OF THE TEST KIT

Materials provided

1. One pouch containing a test and a bag of desiccant.
2. Leaflet with instructions for use.

Materials needed but not provided

A urine collection cup to collect the urine.

STORAGE CONDITIONS AND SHELF LIFE

1. Store in dry places at 35°F - 86°F (2°C - 30°C). Do not freeze.
2. The shelf life is 24 months. Use the test within 1 hour after removing from the foil.
3. See the product package for manufacture date and expiration date.

WARNING AND PRECAUTIONS

1. This product is only for in vitro testing of human urine. Not for internal use.
2. This product is for single-use. Do not reuse.
3. Keep out of the reach of children.
4. The desiccant in the foil pouch is for storage purposes only. Do not swallow.
5. Do not use the test if the pouch is damaged or not well sealed.
6. Do not use the product after its expiration date.

SAMPLE REQUIREMENTS

The urine shall be collected in a urine collection cup. Or directly urinate on the absorbent tip of the test. The first morning urine specimen is preferred, however, urine specimens collected at any time of the day can be used. Please perform the test immediately after urine collection.

TAKE THE TEST

The test should be performed at room temperature (59°F - 86°F / 15°C - 30°C) and avoid strong light.

1. Remove the test from foil, take off the cap and use immediately.

Test in Urine Stream: Hold the test with the absorbent tip pointing downwards into your urine stream (**Fig.2**). **Caution:** Make sure that only the absorbent tip is wet during sampling and that other parts of the test remain dry.

Or Test in a urine collection cup: Collect your urine in a clean, dry container. Immediately after collection, dip more than half of the absorbent tip into the urine (**Fig.3**). **Caution:** Make sure that only the absorbent tip is wet during sampling and that other parts of the test remain dry.

2. The hourglass symbol "⌚" will keep flashing on the display screen (**Fig.4**). **Caution:** Never hold the test with the absorbent tip pointing upwards after sample addition (**Fig.5**).

3. After about ten seconds, you will hear two beeps. Then remove the test from the urine. Lay the test flat with the display screen facing upward. You may put back the cap.

4. Please wait for **5 minutes** and there will be another two beeps, then you can read the results.



Fig.1 Test

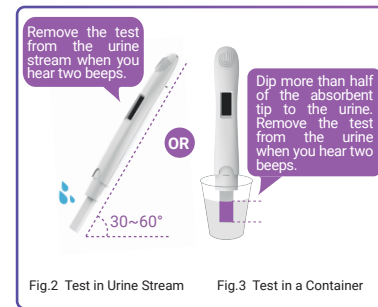


Fig.2 Test in Urine Stream

Fig.3 Test in a Container



Fig.4 Hourglass symbol



Fig.5 Never hold the test with the absorbent tip pointing upwards after sample addition

READ YOUR RESULT

Positive: When the display screen displays "+" (positive), it indicates you are pregnant (See Q1).

Negative: When the display screen displays "-" (negative), it indicates you are not pregnant (See Q2).

Invalid: When the display screen displays "📖" (invalid), it indicates that the test is invalid (See Q5).



Positive



Negative



Invalid

TEST ERRORS

Failures	Failure Analysis	Treatment
Display shows nothing.	No battery charge or the battery is not correctly connected.	Please check the expiration date of the test.
Show test invalid.	The sample volume is not enough or test was not conducted correctly.	Test again with a new test following the instruction carefully.
The test did not beep twice.	Circuit disconnected.	Contact the distributor.

QUESTIONS & ANSWERS

1. My result is Pregnant. What should I do?

The positive test results may indicate pregnancy. See your doctor to confirm, who can advise you on your next steps.

2. My result is Not Pregnant. What does this mean?

When the test result is negative, it indicates that you may not be pregnant. If pregnancy is still suspected, test again using a new Amazewell™ Digital Pregnancy Rapid Test with morning urine 48-72 hours later. If you still get a negative result and your period is late, you should see your doctor.

3. How does Amazewell™ Digital Pregnancy Rapid Test work?

It detects the pregnancy hormone hCG (human Chorionic Gonadotrophin) in urine.

4. Can any medication or conditions affect the results?

Medications containing hCG can affect the result. A number of conditions other than pregnancy cause elevated levels of hCG and could cause misleading results. Unclean urine collection containers or improper operation will also produce incorrect results.

5. How do I know that the test was run properly?

In normal using, an hourglass symbol will appear in the display screen after adding sample. If it does not display on the screen one minute after adding sample, it indicates that the operation is abnormal. It is necessary to retest with a new test. When an open book symbol "📖" appears in the display screen, it indicates that the test is invalid. Please make sure that you have conducted the test correctly according to the instructions, and retest with a new test.

ASSISTANCE

Call toll-free **(888) 695-5248** (Monday – Friday 9:00 am – 5:00 pm, CST) or email to support@docheckusa.com.

Any serious incident related to the device shall be reported to the manufacturer and the competent authority in your country.

MEANING OF SYMBOLS ON PACKAGE



Consult instructions for use



Keep dry



Store at 35°F - 86°F (2°C - 30°C)



Keep away from sunlight



Use-by date



Catalogue number



Batch code



Do not reuse



In vitro diagnostic medical device



Do not use if package is damaged